



QT Cloud USER'S MANUAL

<https://dashboard.qtmedical.com/login>



Web service version v1.14.6

[Contact Us](#)

T: +65 8022 8433

E: hello@ecg-future.com



Version: V2-R3

Update Date : 09/30/2021

Table of content

QT Cloud (Web Service)

1. Registration	2
1.1 Registration Forms	2
1.2 Consent Forms (PCAdmin Account ONLY)	5
2. Account Activation and Setting	7
2.1 Account Activation	7
2.2 Account Information	8
2.3 Password	9
2.4 Notification	12
3. Account Creation (PCAdmin account ONLY)	13
3.1 Physician Creation	13
3.2 Operator Creation	16
4. ECG Records	18
4.1 Overview	18
5. ECG Summary View	20
5.1 Measurement Adjustment (Physician Account ONLY)	20
5.2 Caliper (Physician Account ONLY)	22
5.3 ECG Diagnosis (Physician Account ONLY)	24
6. Patient Profile Management	26
6.1 Overview	26
7. Assign	29
7.1 Patient Profile to ECG	29

Customer Service Support

31



QT Cloud (Web Service)

1. Registration


1.1. Registration Forms

A one-time setup is required before you can begin using the QT ECG system. Have your Bluetooth-enabled mobile device ready and follow the steps below:


Input the following URL in your browser (**we currently only support the "Google Chrome" browser**) to open the QT Medical's dashboard.

<https://dashboard.qtmedical.com/login>


Select **"Activate"**.

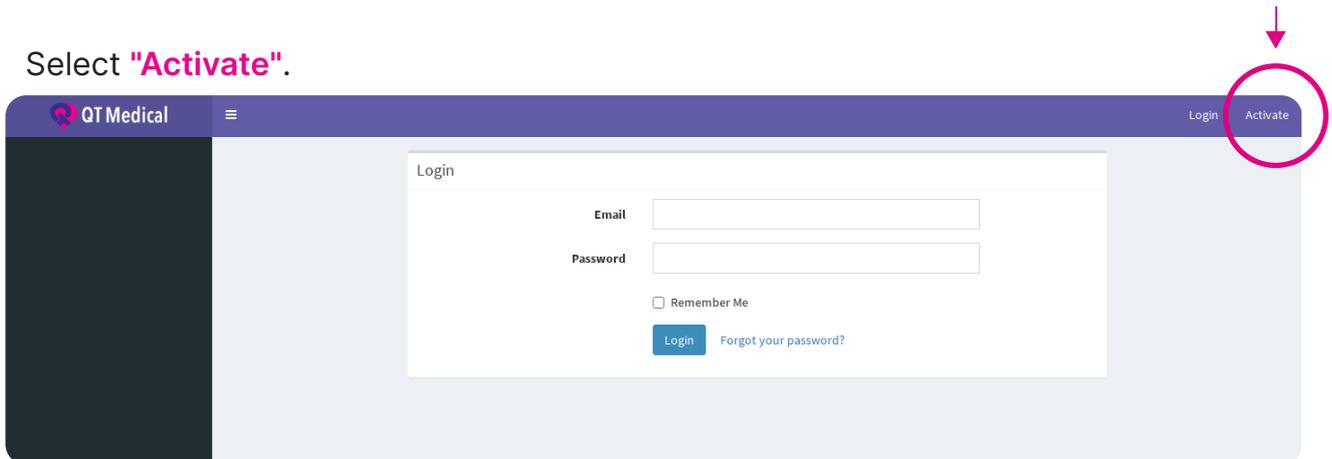


Fill in the information, you can also create physician account at the same time by filling in your Physician Registration Number. Select **"Submit"** after when you complete the information.

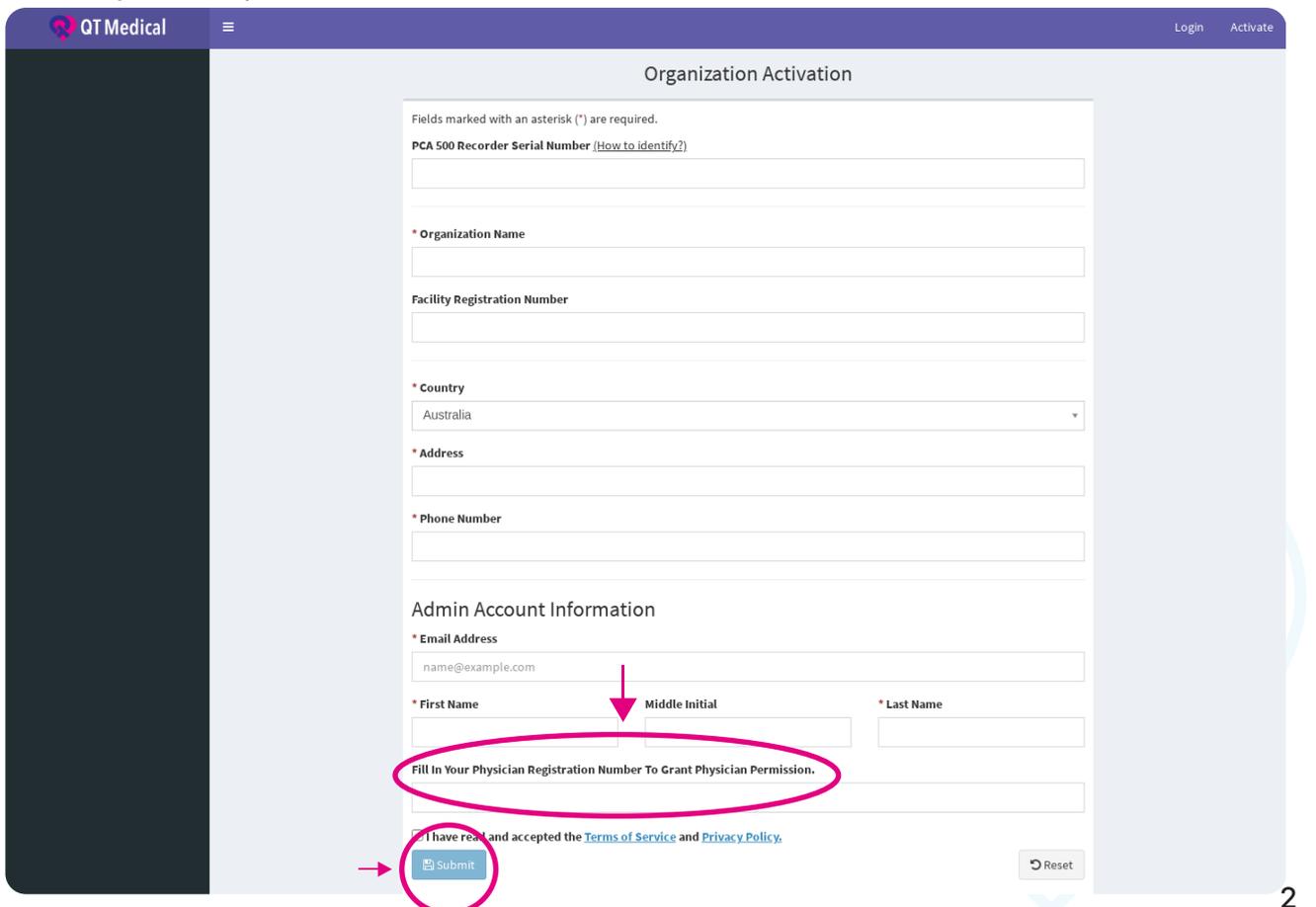


After submitting the information, you will need to **verify your email address**, follow the on-screen instructions.

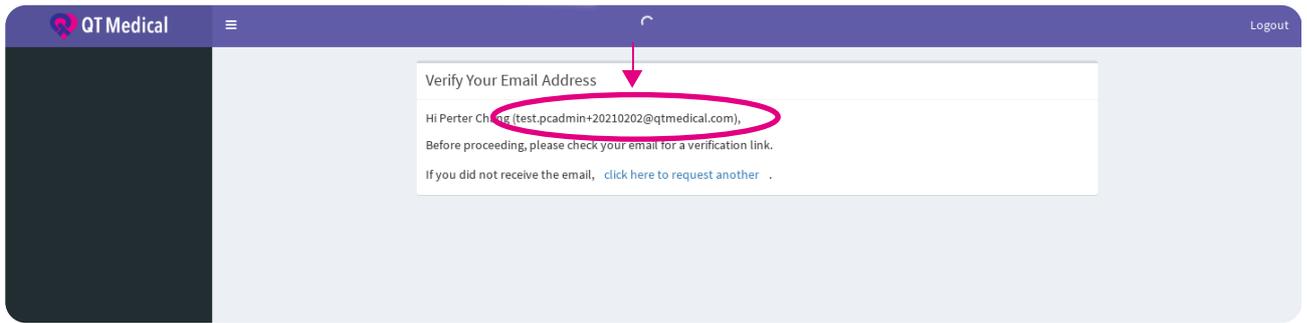



Check your email, and you will get your administrator password (please also check Spam Folder). Click the **"Verify Email Address"** to verify your email:

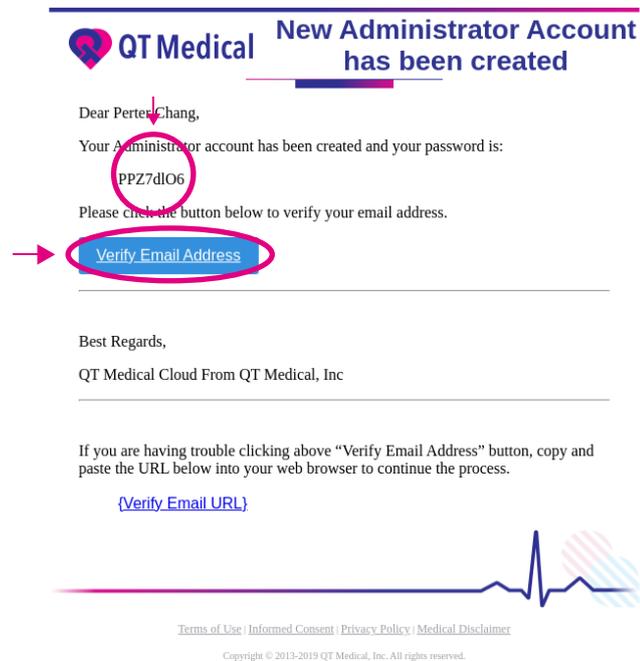

If you are having trouble clicking on the **“Verify Email Address”** button, copy and paste the URL below into your web browser to continue the process.

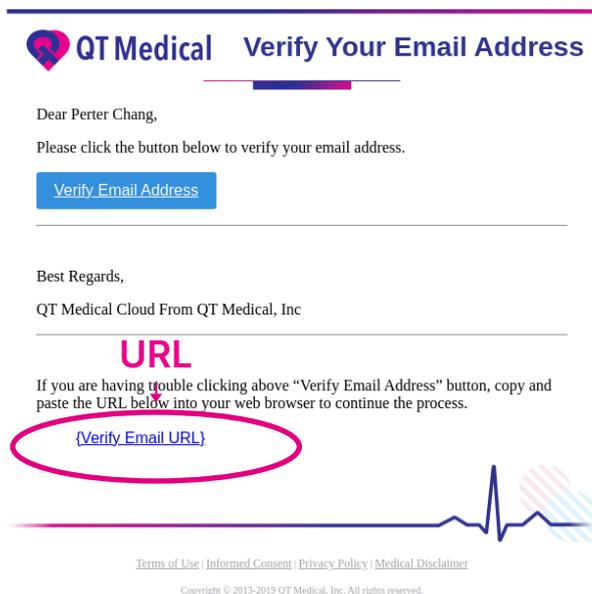


After successful verification, log into your account.

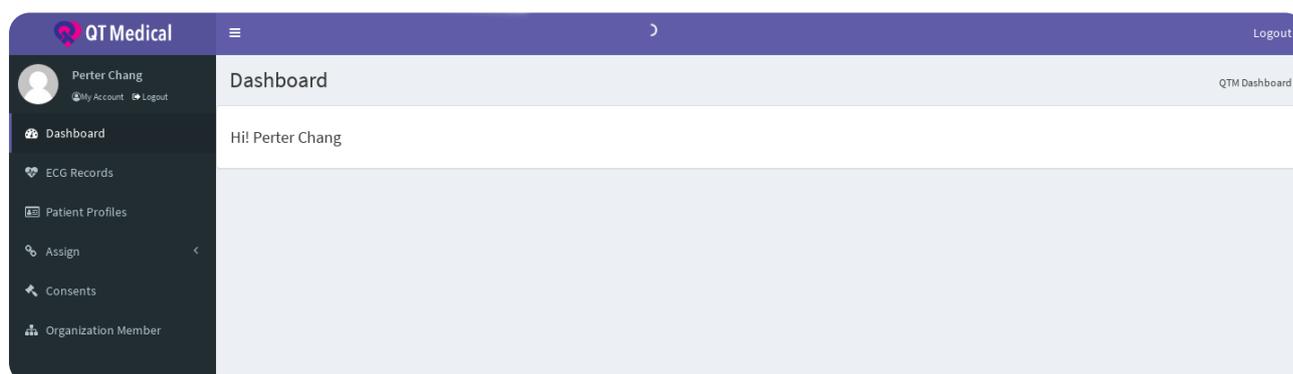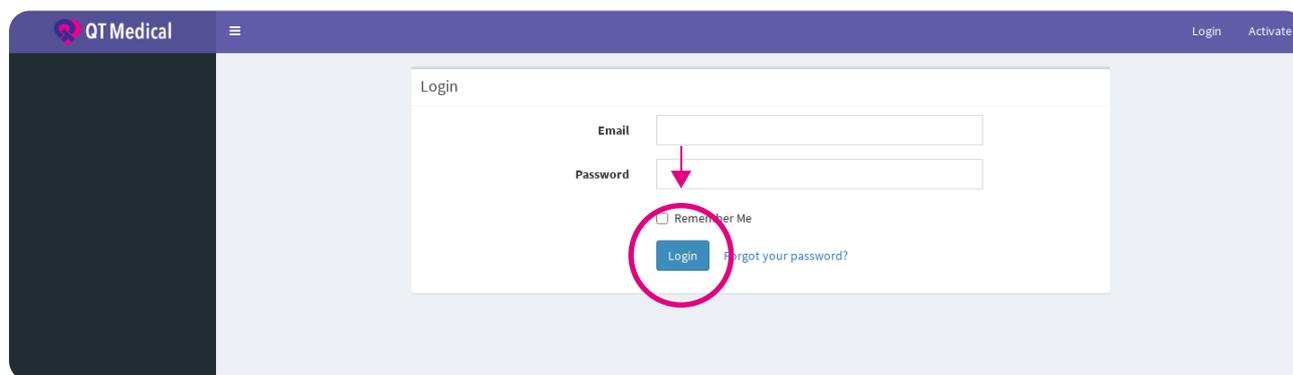


1.2. Consent Forms PCAdmin Account ONLY

When you log in for the first time, you will be prompted to agree to our consents.


Select the tab to read **"Terms of Use"**.

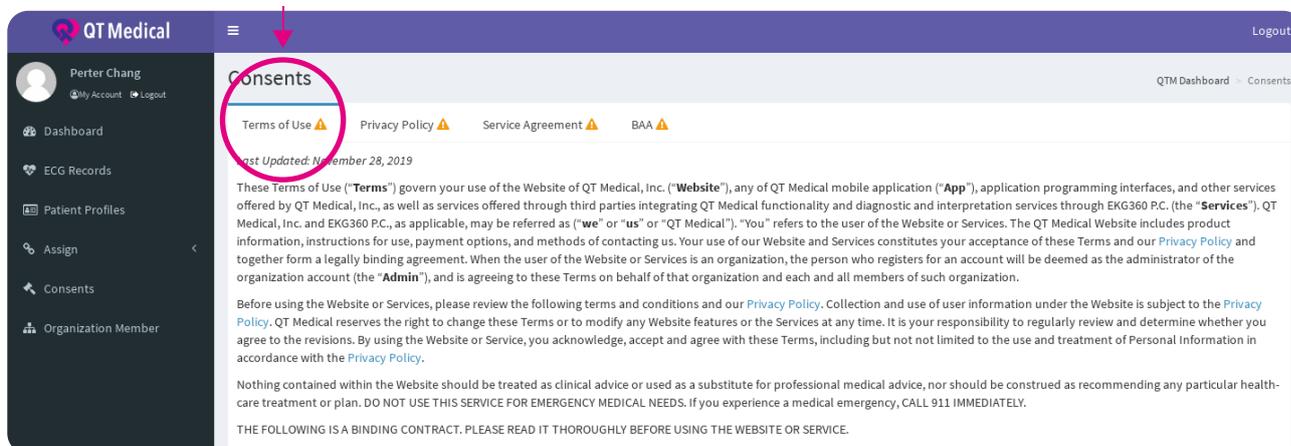

After reading the terms of use, tap **"Agree"**.

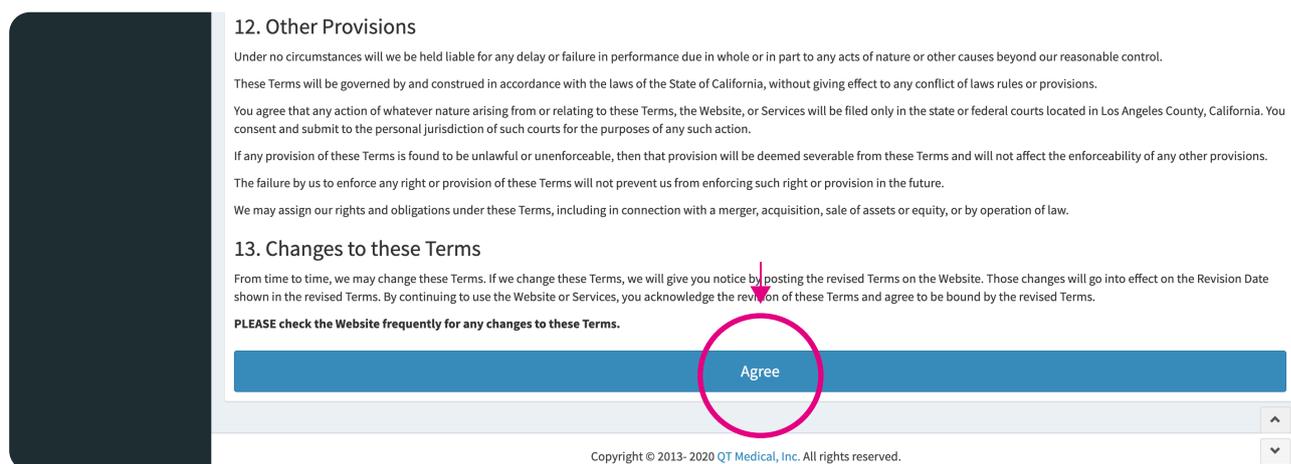

Select **"Privacy Policy"** to read.

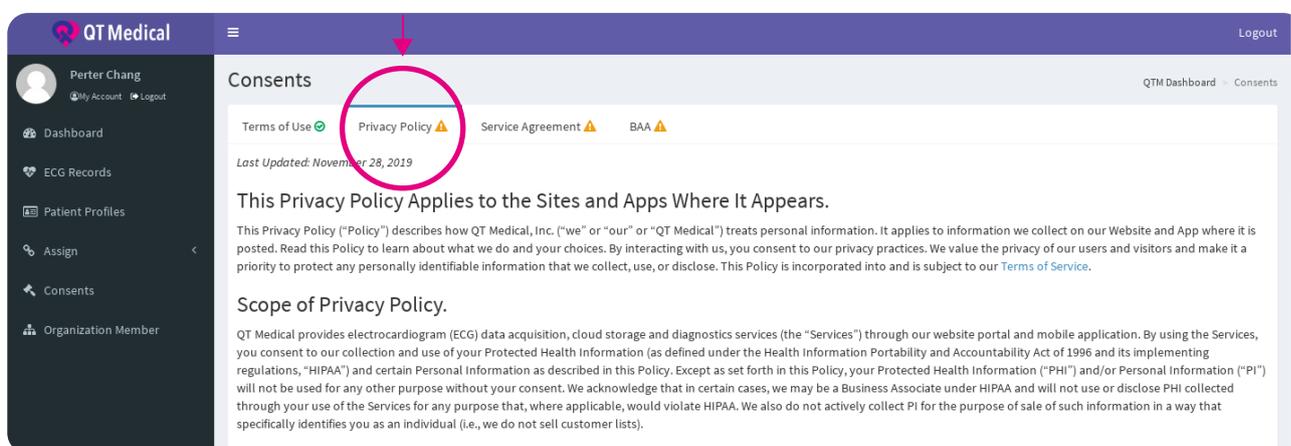

After reading the privacy policy, tap "Agree".

You Can Contact Us About This Policy and Your Information.

If you have any questions about this Privacy Policy or our data practices, please email us at privacy@qtmedical.com. You can also write or call us at:

QT Medical, Inc.
Attn: Brett Chien, PhD
1360 Valley Vista Dr., Suite 203, Diamond Bar, CA 91765
(909)323-0007

We will not take action against you for filing a complaint. If you have a complaint concerning our compliance with applicable privacy laws, we will investigate your complaint and take appropriate measures. You may also file a complaint relating to our use and disclosure of your PHI to the United States Department of Health and Human Services Office for Civil Rights at <https://ocrportal.hhs.gov/ocr/smartscreen/main.jsf>.

We May Update This Policy.

This Policy may be updated from time to time without further notice to you and this will be reflected by a "Last modified" date above. Please revisit this webpage regularly for any changes. By continuing to use the Services, you are consenting to the terms of the then-current Policy.

Agree

Copyright © 2013- 2020 QT Medical, Inc. All rights reserved.

After agreeing to all consents, you can see the summary view after logging in successfully.

QT Medical

Perter Chang
My Account Logout

Dashboard
ECG Records
Patient Profiles
Assign
Consents
Organization Member

Consents

Terms of Use Privacy Policy Service Agreement BAA

Last Updated: November 28, 2019

These Terms of Use ("Terms") govern your use of the Website of QT Medical, Inc. ("Website"), any of QT Medical mobile application ("App"), application programming interfaces, and other services offered by QT Medical, Inc., as well as services offered through third parties integrating QT Medical functionality and diagnostic and interpretation services through EKG360 P.C. (the "Services"). QT Medical, Inc. and EKG360 P.C., as applicable, may be referred as ("we" or "us" or "QT Medical"). "You" refers to the user of the Website or Services. The QT Medical Website includes product information, instructions for use, payment options, and methods of contacting us. Your use of our Website and Services constitutes your acceptance of these Terms and our Privacy Policy and together form a legally binding agreement. When the user of the Website or Services is an organization, the person who registers for an account will be deemed as the administrator of the organization account (the "Admin"), and is agreeing to these Terms on behalf of that organization and each and all members of such organization.

Before using the Website or Services, please review the following terms and conditions and our Privacy Policy. Collection and use of user information under the Website is subject to the Privacy Policy. QT Medical reserves the right to change these Terms or to modify any Website features or the Services at any time. It is your responsibility to regularly review and determine whether you agree to the revisions. By using the Website or Service, you acknowledge, accept and agree with these Terms, including but not limited to the use and treatment of Personal Information in accordance with the Privacy Policy.

Nothing contained within the Website should be treated as clinical advice or used as a substitute for professional medical advice, nor should be construed as recommending any particular health-care treatment or plan. DO NOT USE THIS SERVICE FOR EMERGENCY MEDICAL NEEDS. If you experience a medical emergency, CALL 911 IMMEDIATELY.

THE FOLLOWING IS A BINDING CONTRACT. PLEASE READ IT THOROUGHLY BEFORE USING THE WEBSITE OR SERVICE.

QT Medical

Perter Chang
My Account Logout

Dashboard
ECG Records
Patient Profiles
Assign
Consents

Dashboard


Hi! Perter Chang





2. Account Activation and Setting

2.1. Account Activation

Check your email, and you will get your password. Click the **"Verify Email Address"** to verify your email:

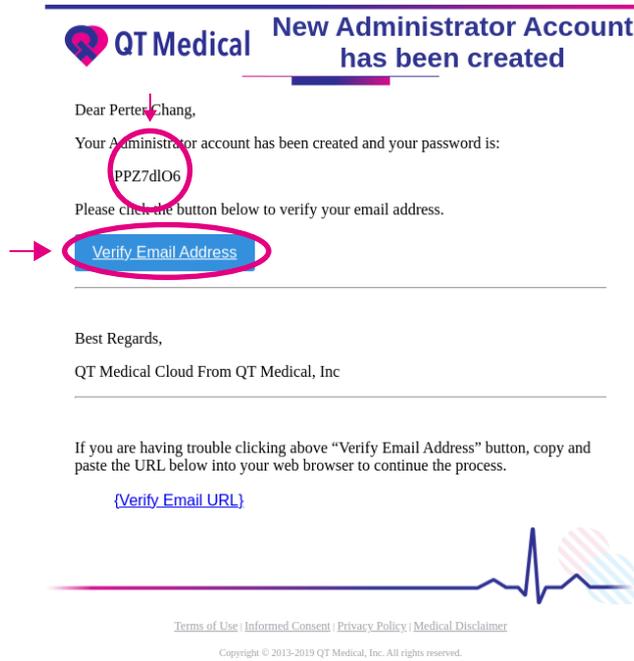


After successful verification, log into your account.

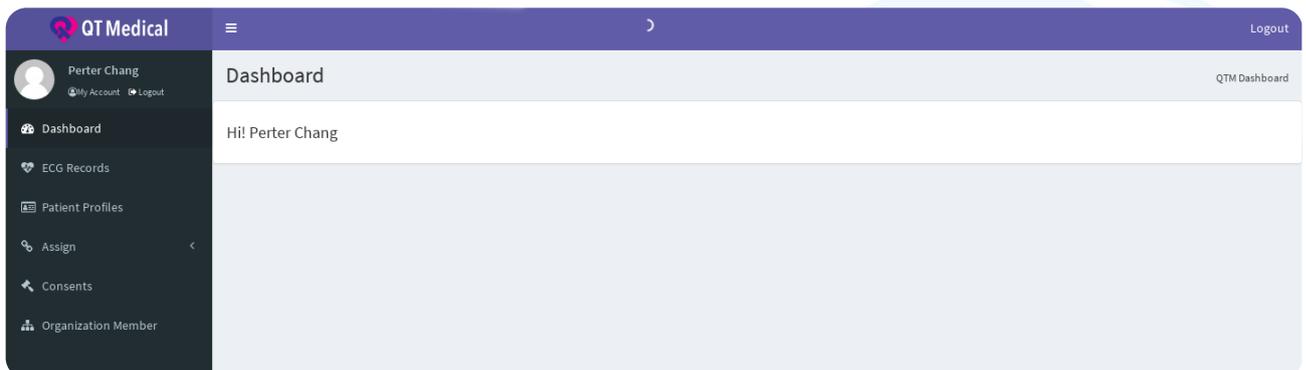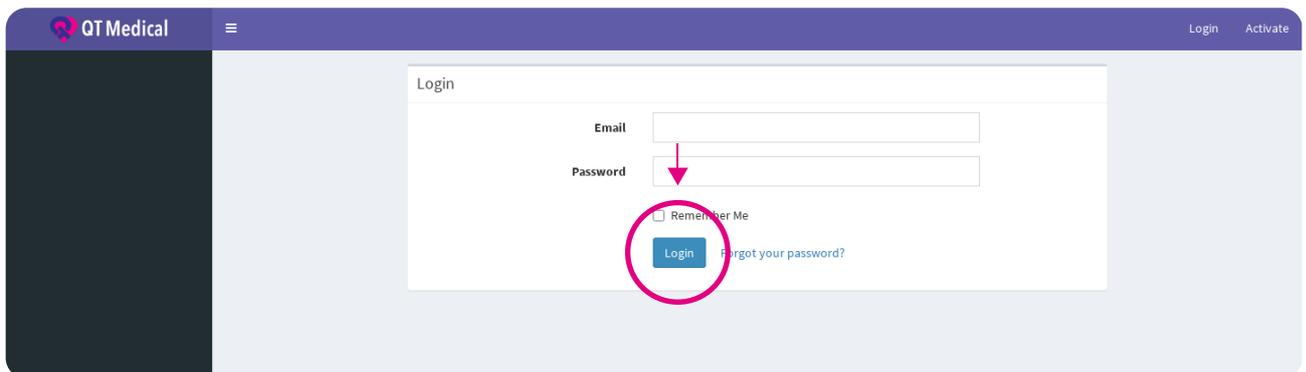


2.2. Account Information

2.2.1 Update the account info.

Click **"My account"**.

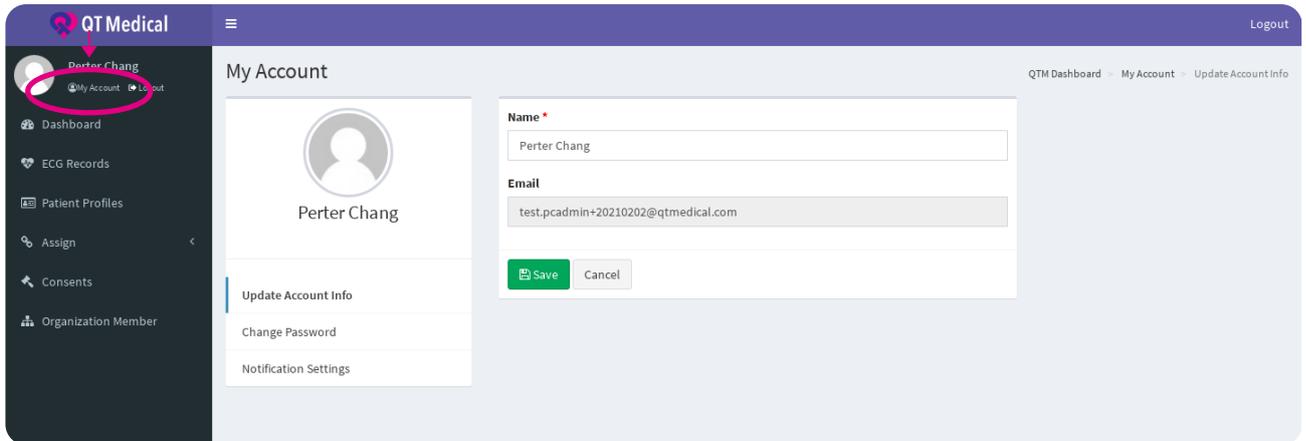


To change your account info, please select **"Update Account Info"**.
Edit your info, click the Save button, you will see the **"Success"** popup .

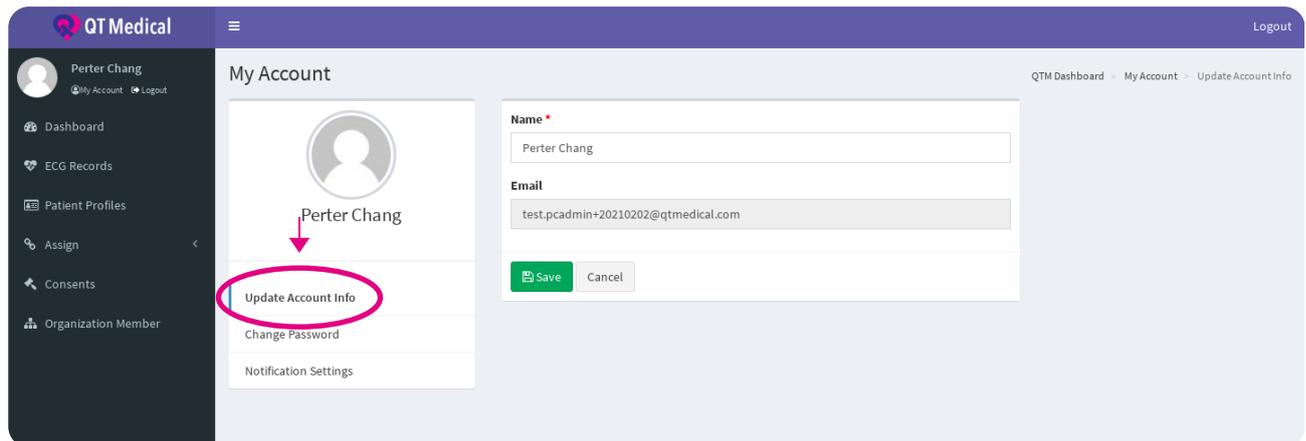

2.3. Password


2.3.1 Forget password

Click the **"Forgot your Password?"** .

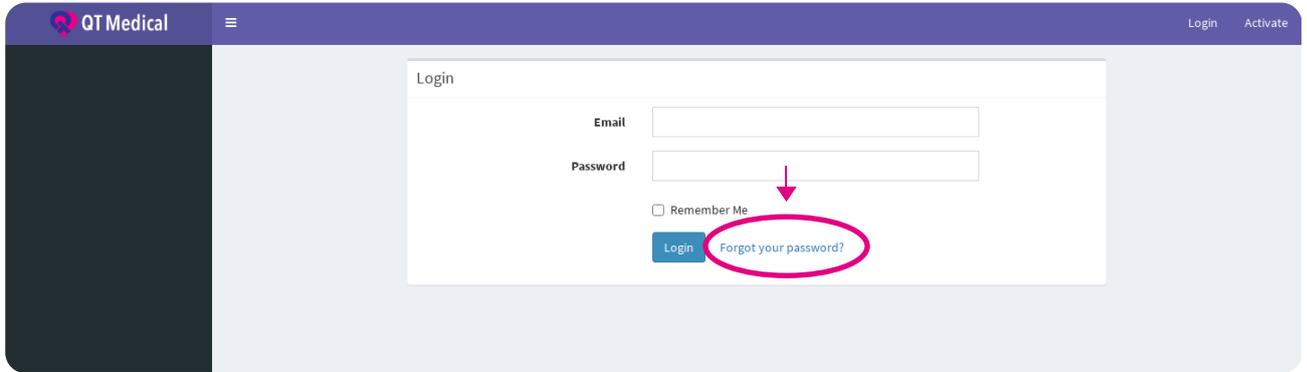


The screenshot shows the QT Medical login interface. At the top left is the QT Medical logo, and at the top right are 'Login' and 'Activate' links. The main content area is titled 'Login' and contains an 'Email' input field, a 'Password' input field, a 'Remember Me' checkbox, a 'Login' button, and a 'Forgot your password?' link. A pink arrow points to the 'Forgot your password?' link, which is circled in pink.

Enter your Email address and click the **"Reset Password"**.

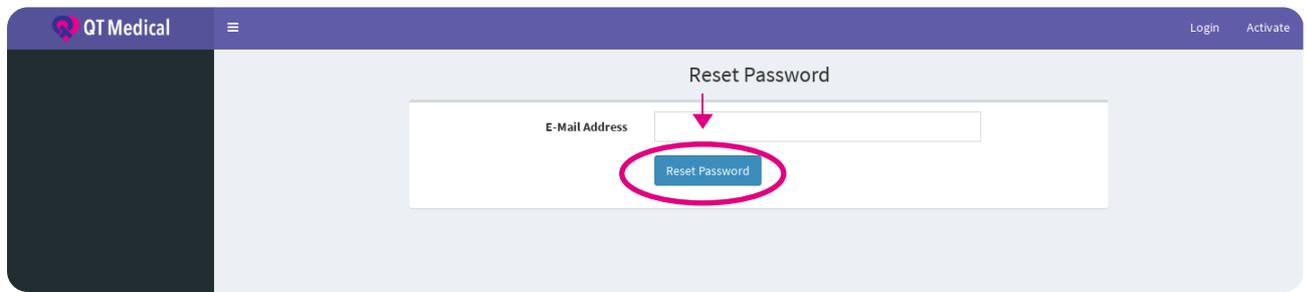


The screenshot shows the 'Reset Password' page. At the top left is the QT Medical logo, and at the top right are 'Login' and 'Activate' links. The main content area is titled 'Reset Password' and contains an 'E-Mail Address' input field and a 'Reset Password' button. A pink arrow points to the 'Reset Password' button, which is circled in pink.

Check your email, you will get the **"reset password notification"**.

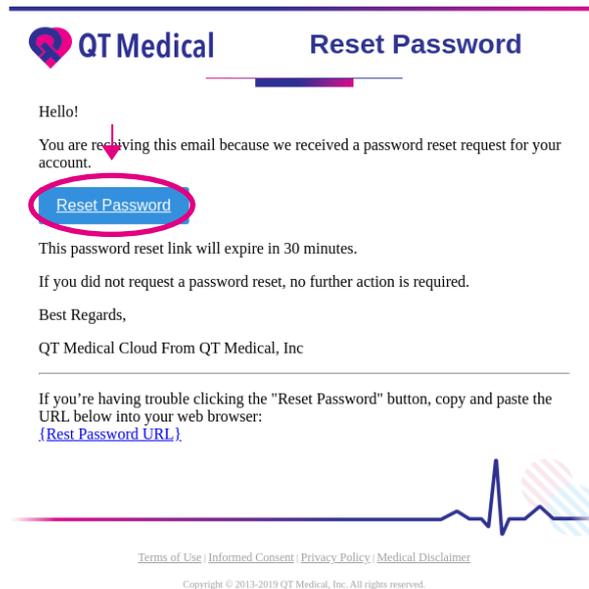

Enter your Email and new password and click **"Reset Password"**.
You will see the "Success" message popup .

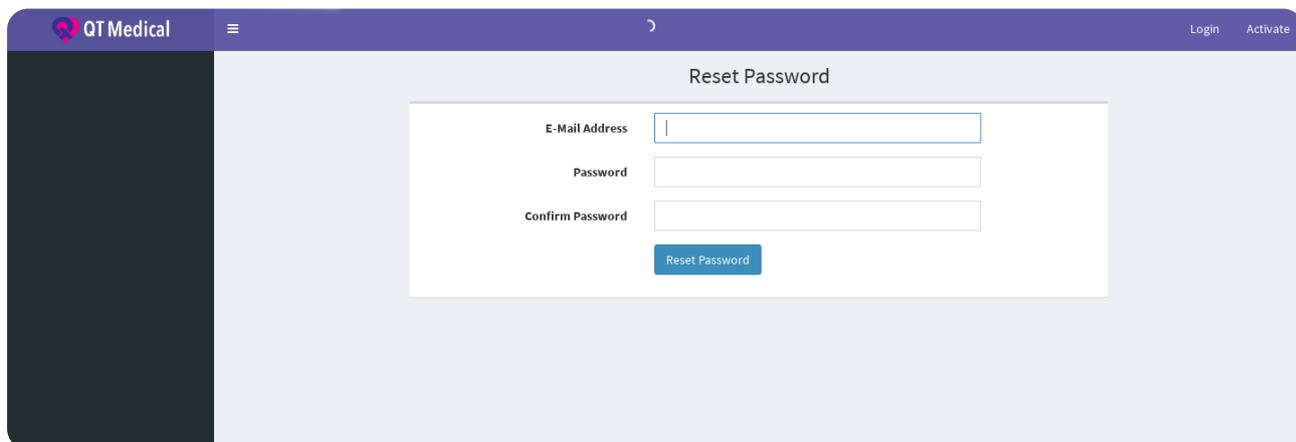


The screenshot shows a web application interface for "QT Medical". The header is dark purple with the logo on the left and "Login" and "Activate" links on the right. The main content area is light blue and features a "Reset Password" form. The form is a white box with three input fields: "E-Mail Address", "Password", and "Confirm Password". Below the fields is a blue button labeled "Reset Password".




2.3.2 Change password

Select **"Change Password"** to set your new password.

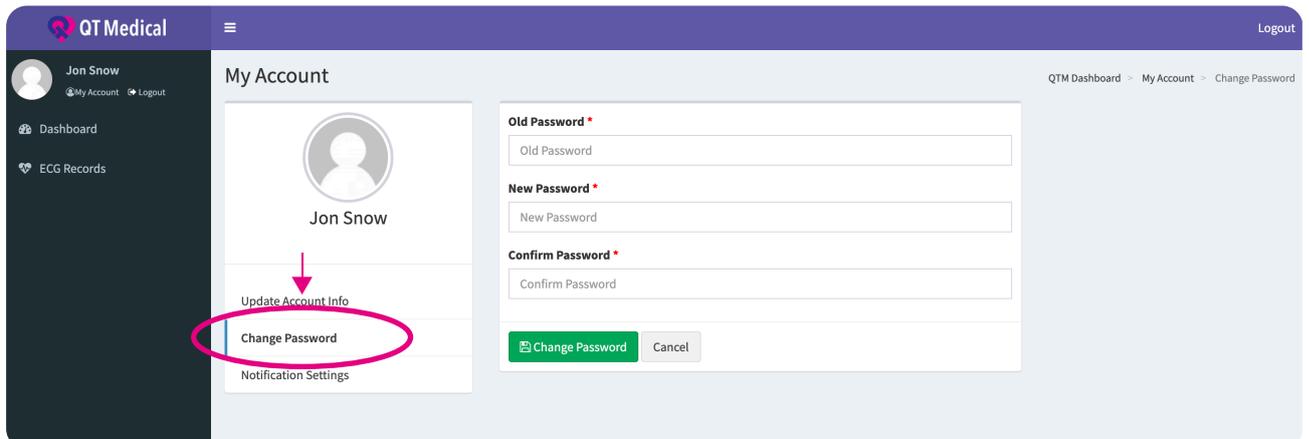

The screenshot shows the 'My Account' page for user Jon Snow. On the left, a navigation menu includes 'Dashboard' and 'ECG Records'. The main content area shows a profile card for Jon Snow with three options: 'Update Account Info', 'Change Password', and 'Notification Settings'. The 'Change Password' option is circled in pink. A pink arrow points from this option to the 'Change Password' button on the form. The form contains three input fields: 'Old Password', 'New Password', and 'Confirm Password', followed by a green 'Change Password' button and a grey 'Cancel' button.

After setting your new password, please select **"Change Password"**.

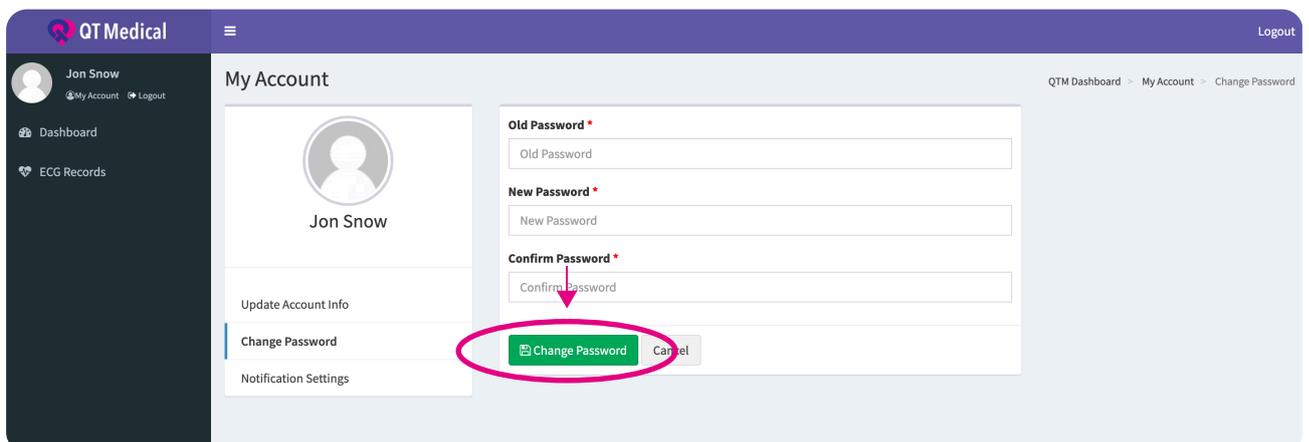


This screenshot shows the same 'My Account' page. The 'Change Password' option in the account menu is no longer circled. Instead, the 'Change Password' button on the form is circled in pink. A pink arrow points from the 'Confirm Password' input field to this button.




2.4. Notification Settings

Click the **"Notification Settings"** and you can select the options below.

Physician Account ONLY : Gets immediate notification when ECG Uploaded.

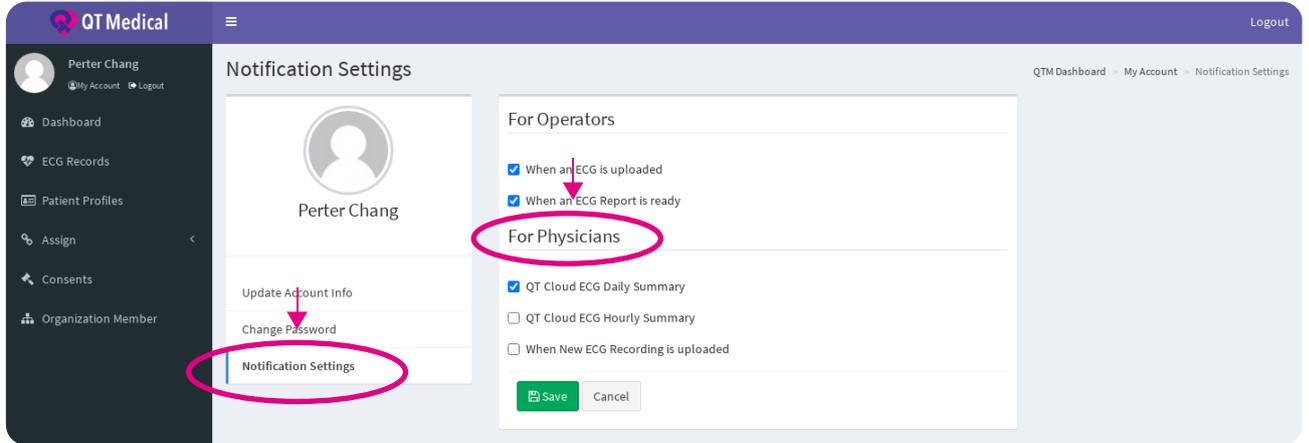


You will receive your notifications.

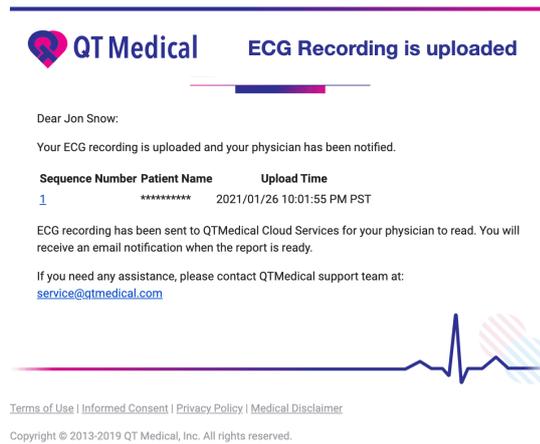


3. Accounts Creation PCAdmin Account ONLY

3.1 Physician Account Creation

Please select **"Organization Member"** to create and edit accounts for your organization.

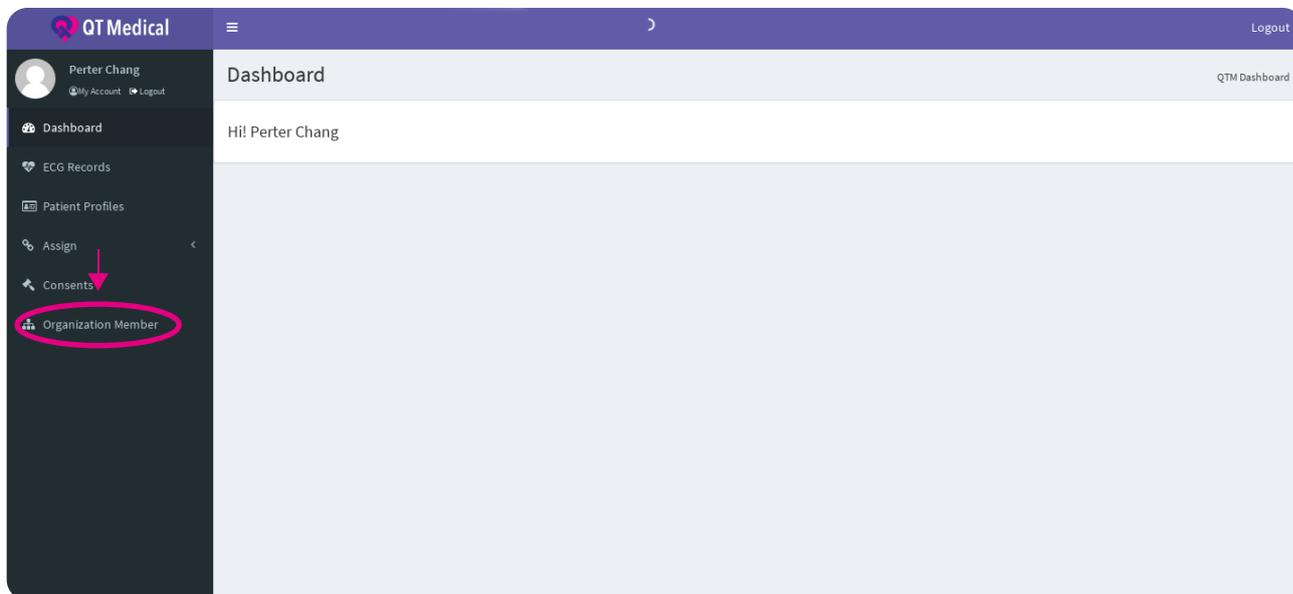

Fill in the information, you can create two types of accounts, physician or operator, and select **"Submit"** after when you complete the information.

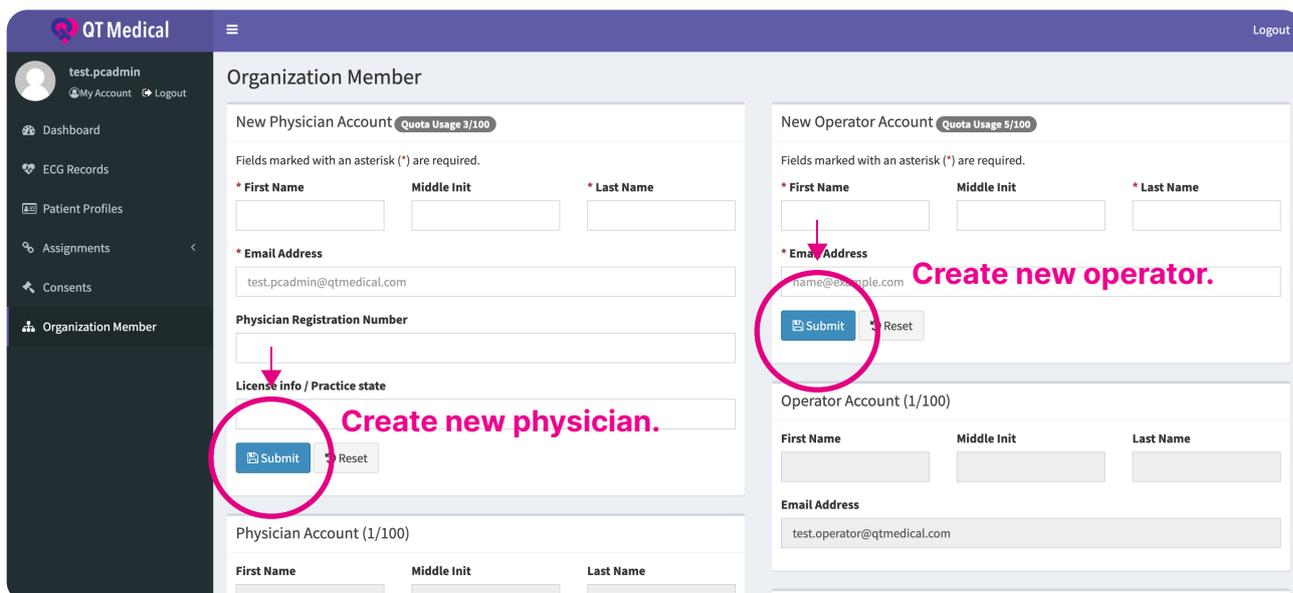

After submitting the information, the message will be shown below. Inform the email's owner to verify the email address and follow the on-screen instructions.

Organization Member

test.pcadmin

My Account Logout

Dashboard

ECG Records

Patient Profiles

Assignments

Consents

Organization Member

Organization Member

Quota Usage 4/100

Fields marked with an asterisk (*) are required.

* First Name Middle Init * Last Name

* Email Address

test.pcadmin@qtmedical.com

Physician Registration Number

License info / Practice state

Submit Reset

Quota Usage 5/100

Fields marked with an asterisk (*) are required.

* First Name Middle Init * Last Name

* Email Address

name@example.com

Submit Reset

Operator Account (1/100)

First Name Middle Init Last Name

Email Address

test.operator@qtmedical.com

Operator Account (2/100)

First Name Middle Init Last Name

Email Address

test.physician@qtmedical.com

Operator Account (3/100)

First Name Middle Init Last Name

Physician Account (1/100)

First Name Middle Init Last Name

Test Web

Email Address (Not Verified)

yt.pai+20210127@qtmedical.com

Physician Registration Number

License info / Practice state

Physician Account (2/100)

First Name Middle Init Last Name

Check the physician's email for the password (please also check Spam Folder). Click the Verify Email Address to verify the email:

QT Medical

New Operator Account has been created

Dear Kent Xu,

Your Operator account has been created and your password is:

Your password cE4dBatY

Please click the button below to verify your email address.

Verify Email Address

Best Regards,

QT Medical Cloud From QT Medical, Inc

If you are having trouble clicking above "Verify Email Address" button, copy and paste the URL below into your web browser to continue the process.

<https://dev-dashboard.qtmedical.com/email/verify/196a09d3-e714-4cd8-8399-109ac84ce3d7/a5e6d5e317ebaeddede449087d616a34d7c3056b7?expires=1614319702&signature=748eaf5dca5d3a25c1a608866f3a183c2b32fb12d55c2bcc03500177ceb8964>


Terms of Use | Informed Consent | Privacy Policy | Medical Disclaimer

Copyright © 2013-2019 QT Medical, Inc. All rights reserved.


u are subscribed to the Google Groups "test.operator" group.
p receiving emails from it, send an email to test.operator+unsubscribe@qtmedical.com.
operator@qtmedical.com
<https://groups.google.com/a/qtmedical.com/msgid/test.operator/68808f299c6c04b9f05142b8341c3b9c%40swift.generated>

Forward

If you are having trouble clicking on the **“Verify Email Address”** button, copy and paste the URL below into your web browser to continue the process.

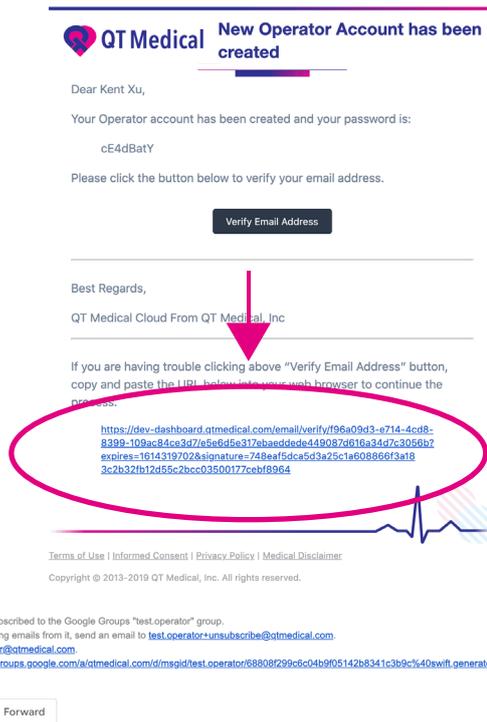


After verifying the email address, you will see the summary view.

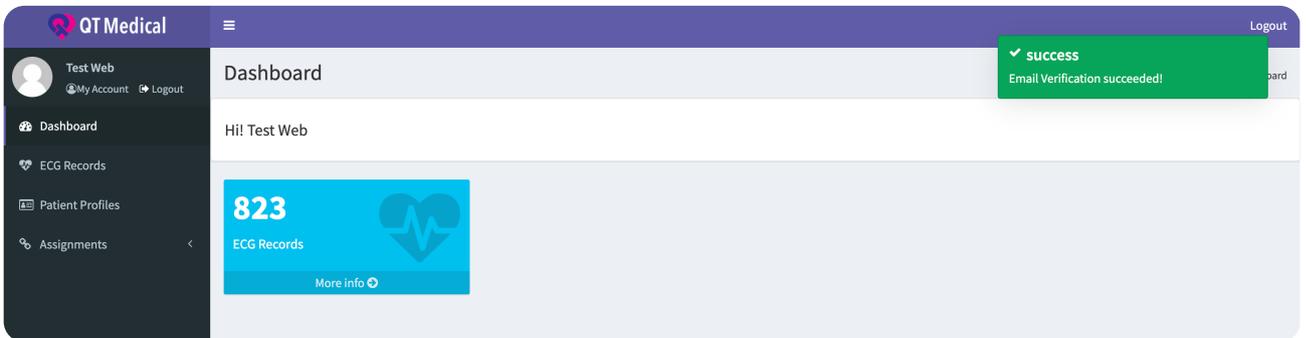


3.2 Operator account Creation

Fill in the information, you can create two types of accounts, and select "Submit" after when you complete the information.

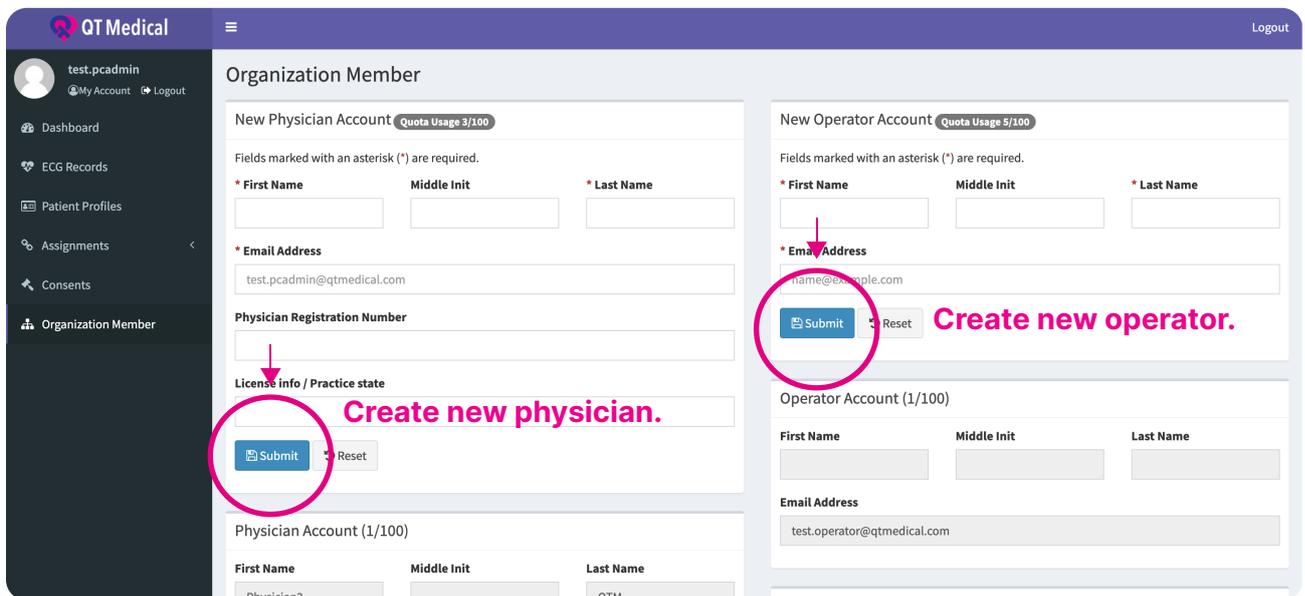


The screenshot shows the 'Organization Member' page in the QT Medical system. It features two main forms: 'New Physician Account' (Quota Usage 3/100) and 'New Operator Account' (Quota Usage 5/100). Both forms have fields for First Name, Middle Init, Last Name, and Email Address. The 'New Operator Account' form also has a 'Submit' button circled in pink, with a pink arrow pointing to it and the text 'Create new operator.' The 'New Physician Account' form also has a 'Submit' button circled in pink, with a pink arrow pointing to it and the text 'Create new physician.'

After submitting the information, the message will be shown below. Inform the email's owner to verify the email address and follow the on-screen instructions.

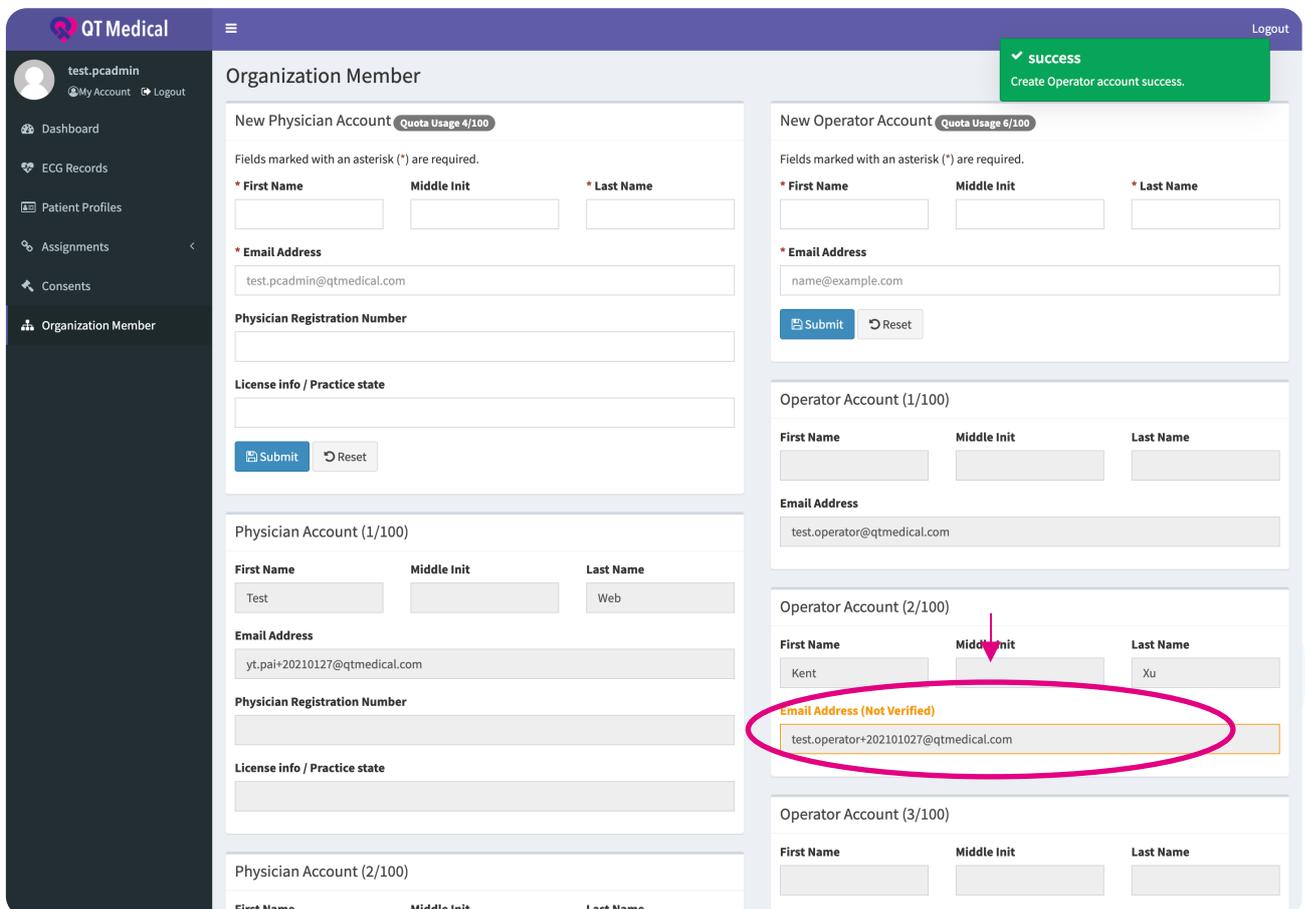


The screenshot shows the 'Organization Member' page after account creation. A green success message 'Create Operator account success.' is displayed at the top right. The 'New Operator Account' form is now filled with 'name@example.com' and has a 'Submit' button. Below it, the 'Operator Account (1/100)' form is filled with 'test.operator@qtmedical.com'. The 'Operator Account (2/100)' form is also filled, with the 'Email Address' field 'test.operator+202101027@qtmedical.com' circled in pink. A pink arrow points to the 'Middle Init' field of the 'Operator Account (2/100)' form.


Check operator's email for the password (please also check Spam Folder). Click the Verify Email Address to verify the email:

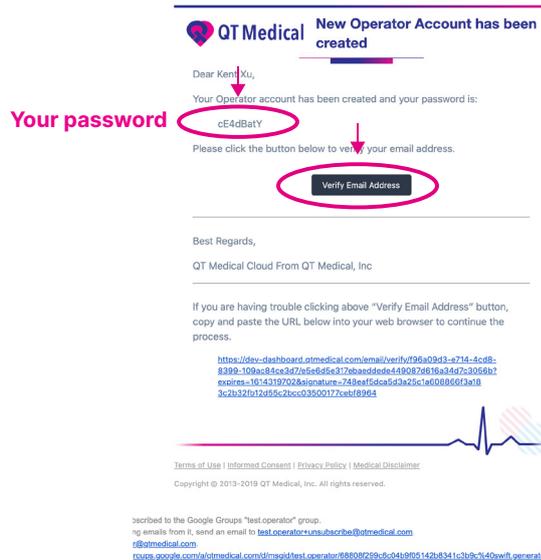

If you are having trouble clicking on the **“Verify Email Address”** button, copy and paste the URL below into your web browser to continue the process.

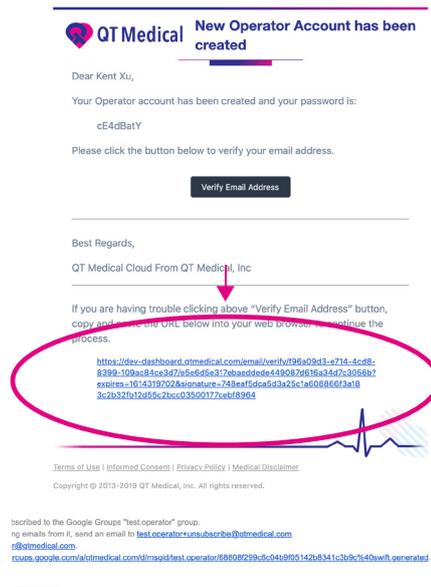

After verifying the email address, you will see the summary view.

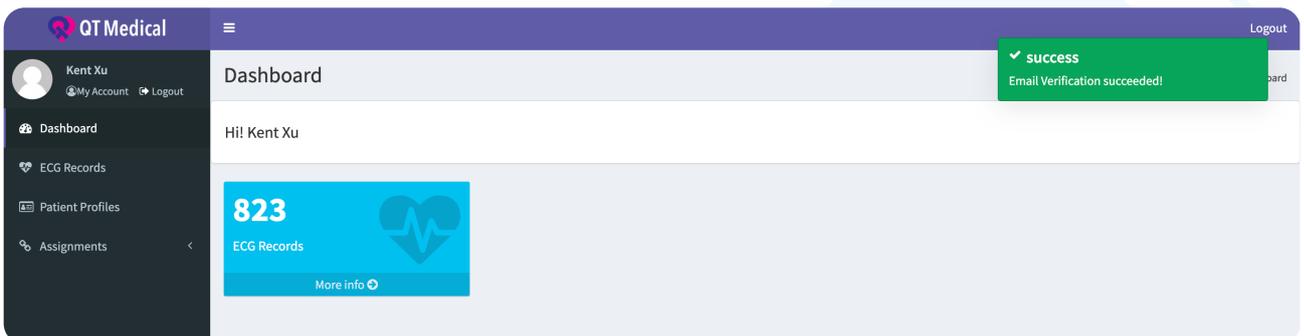

4. ECG Records

4.1 Overview

4.1.1 Tab the **"ECG Records"**, you can see all uploaded ECG records from the dashboard.

The screenshot shows the QT Medical dashboard with the 'ECG Records' tab selected. The sidebar menu has 'ECG Records' highlighted with a pink circle. The main area displays a table of ECG records. The table has the following columns: Session, Patient (MRN, First Name, Last Name, Time Zone), Recorded On, Uploaded On, Status, and Actions. The records are as follows:

Session	MRN	First Name	Last Name	Time Zone	Recorded On	Uploaded On	Status	Actions
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread	Note Summary Report
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Jamaica	01/06/2018 05:12	02/02/2021 04:14	Unread	Note Summary Report
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread	Note Summary Report
3f3af8eb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed	Note
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:10	02/02/2021 03:14	Pending	Note

ECG Records recorded during the same session will be grouped as shown below.

The screenshot shows the QT Medical dashboard with the 'ECG Records' tab selected. The 'Session' column in the table is circled in pink, highlighting that records with the same session ID are grouped together. The records are as follows:

Session	MRN	First Name	Last Name	Time Zone	Recorded On	Uploaded On	Status	Actions
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread	Note Summary Report
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Jamaica	01/06/2018 05:12	02/02/2021 04:14	Unread	Note Summary Report
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread	Note Summary Report
3f3af8eb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed	Note
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:10	02/02/2021 03:14	Pending	Note

4.1.2 Status

- **Unread:** Your report hasn't been diagnosed.
- **Finalized:** Your report have been diagnosed.
- **Failed:** If status shows "Failed", please contact customer service.

hello@ecg-future.com

The screenshot shows the QT Medical dashboard with the 'ECG Records' tab selected. The 'Status' column in the table is circled in pink, highlighting the different status values. The records are as follows:

Session	MRN	First Name	Last Name	Time Zone	Recorded On	Uploaded On	Status	Actions
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread	Note Summary Report
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Jamaica	01/06/2018 05:12	02/02/2021 04:14	Unread	Note Summary Report
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread	Note Summary Report
3f3af8eb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed	Note
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:10	02/02/2021 03:14	Pending	Note
6c2da029	MRN-bd5f2e2f-66771	Colton	Berge	Asia/Samarkand	04/02/2013 06:57	02/02/2021 14:14	Processing	Note
933376eb	MRN-bd5f2e2f-66771	Colton	Berge	Europe/Zagreb	01/05/1975 13:03	02/02/2021 10:14	Finalized	Note Summary Report

4.1.3 You can "Search", and turn on the option to show only Selected Records in a Session.

The screenshot shows the QT Medical ECG Records page. A search bar is located at the top right, circled in pink and labeled "Search". On the left side, there is a toggle switch for "Only Selected Records in Session", which is turned on and circled in pink, with the word "Selected" written next to it. Below the toggle is a table of ECG records with columns for Session, MRN, Patient (First Name, Last Name), Time Zone, Recorded On, Uploaded On, Status, and Actions. The first row of the table is highlighted.

Session	MRN	Patient	Time Zone	Recorded On	Uploaded On	Status	Actions
		First Name	Last Name				
2349b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread Note Summary Report
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread Note Summary Report
3f3a8eb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed Note
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:10	02/02/2021 03:14	Pending Note
6c2da029	MRN-bd5f2e2f-66771	Colton	Berge	Asia/Samarkand	04/02/2013 06:57	02/02/2021 14:14	Processing Note

4.1.4 Click on "Note", to view the Note in the popup window.

This screenshot is similar to the previous one, but the "Note" button in the Actions column of the first row is circled in pink. The "Only Selected Records in Session" toggle is also visible and turned on.


This screenshot shows the same interface as the previous one, but with a popup window open. The popup window has a title "Note for ECG ID: a78aa272-695f-4b46-92c0-2e88486a332f" and contains the text "This is the correct record." The background of the interface is dimmed.





5. ECG Summary View

5.1 Measurements Adjustment (Physician Account ONLY)


5.1.1 Click on the "ECG Records", you can see all uploaded ECG records from the dashboard.

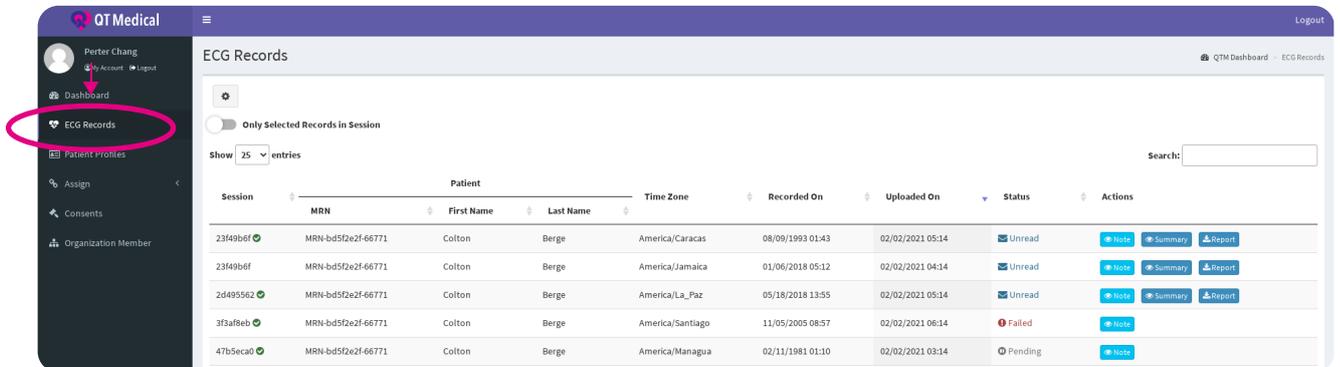


Click on the "Summary", you can see the summary view.

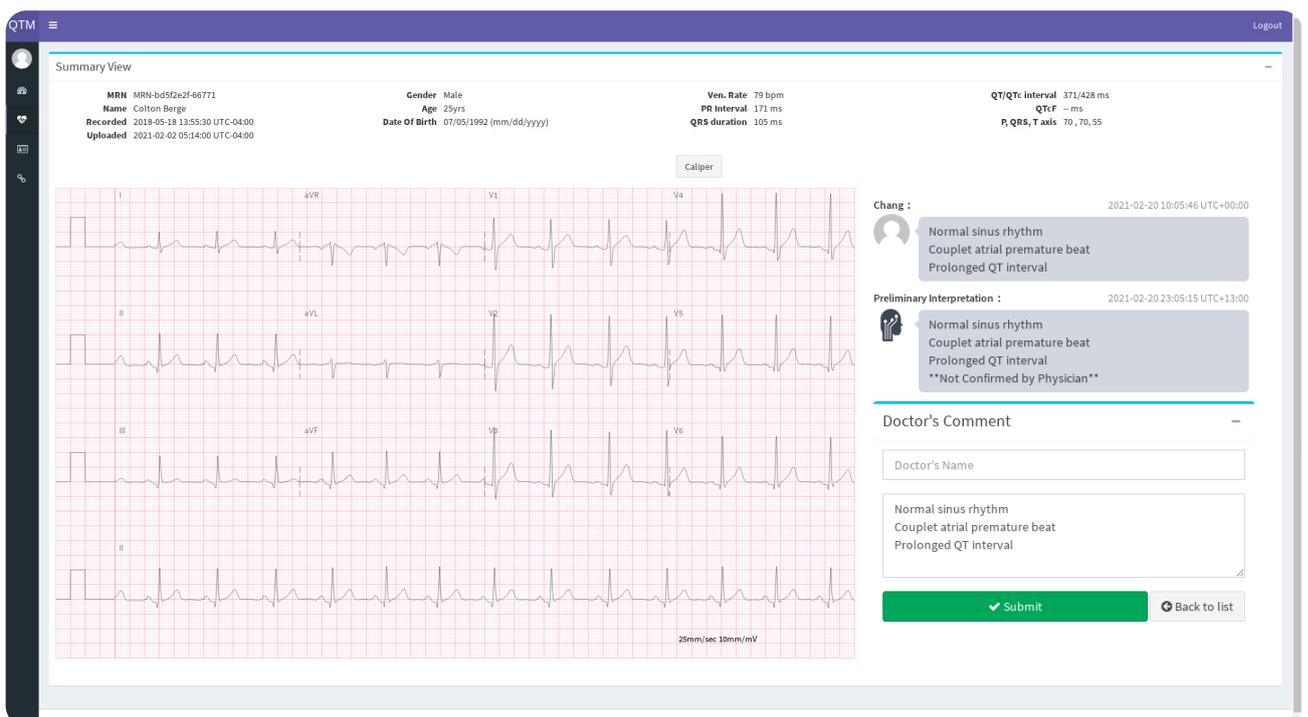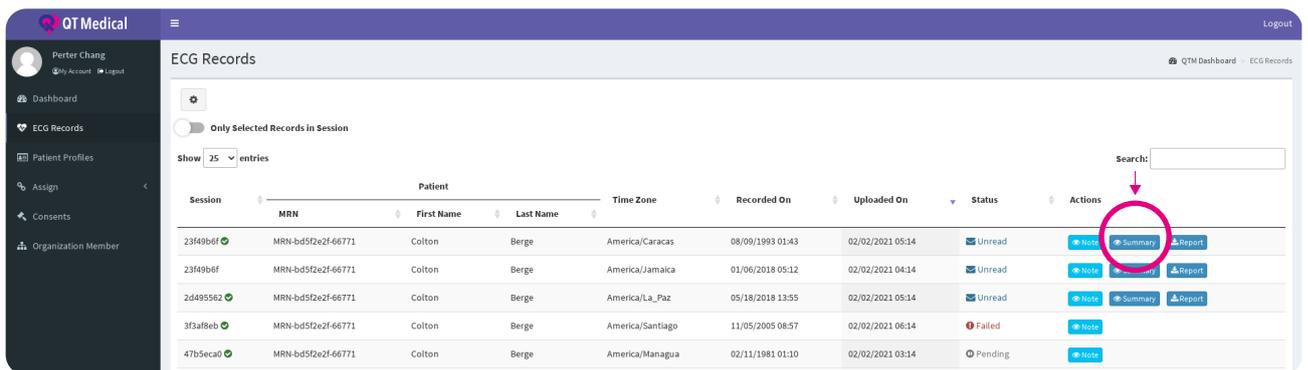


If you want to change the measurement, click the "numerical value".

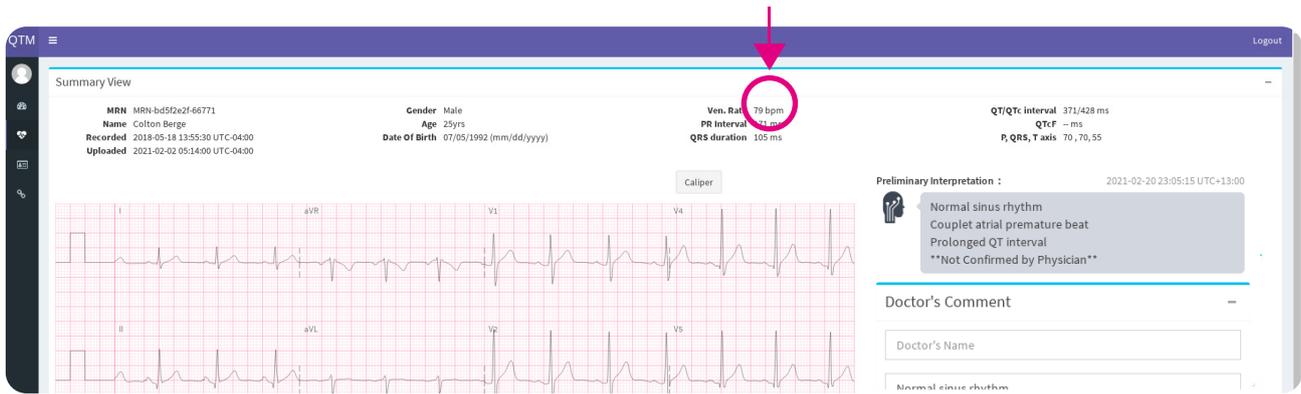

You will see the edit bar.

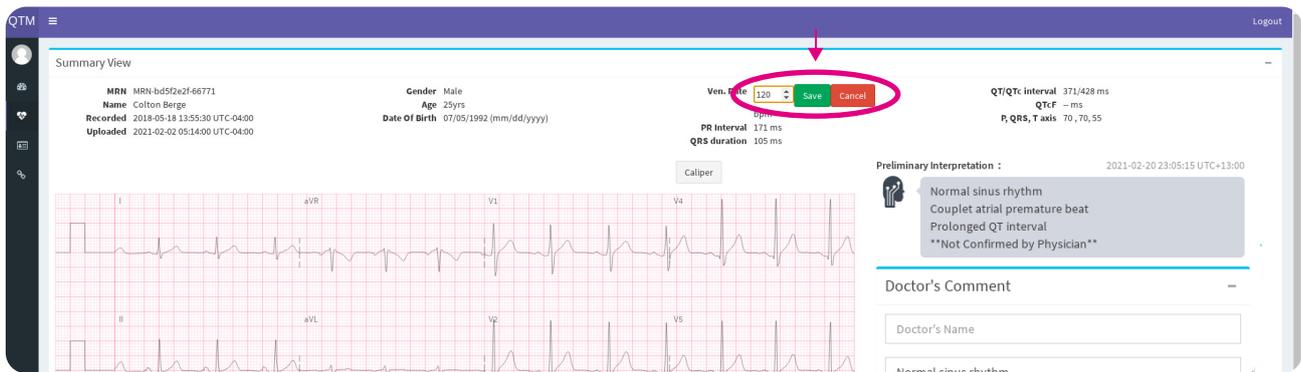

After editing, the numerical value will become bold.

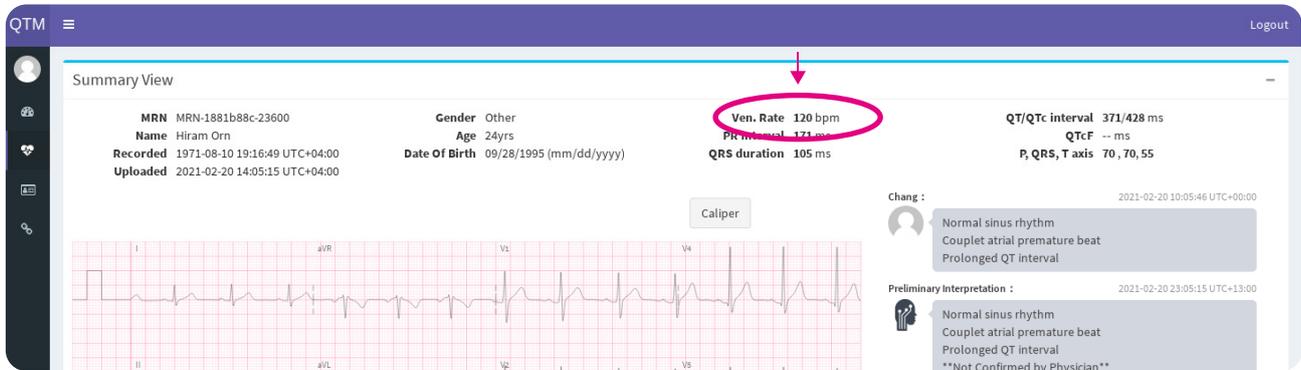

5.2 Caliper (Physician Account ONLY)

Select "Caliper" to show the movable green line and blue lines.

Summary View

MRN: MRN-1881b88c-23600
Name: Hiram Orn
Recorded: 1971-08-10 19:16:49 UTC+04:00
Uploaded: 2021-02-20 14:05:15 UTC+04:00

Gender: Other
Age: 24yrs
Date Of Birth: 09/28/1995 (mm/dd/yyyy)

Ven. Rate: 120 bpm
PR Interval: 171 ms
RS duration: 105 ms

QT/QTc interval: 371/428 ms
QTcF: -- ms
P, QRS, T axis: 70, 70, 55

Caliper

Chang : 2021-02-20 10:05:46 UTC+00:00
Normal sinus rhythm
Couplet atrial premature beat
Prolonged QT interval

Preliminary Interpretation : 2021-02-20 23:05:15 UTC+13:00
Normal sinus rhythm
Couplet atrial premature beat
Prolonged QT interval
Not Confirmed by Physician

Doctor's Comment

Doctor's Name

Normal sinus rhythm
Couplet atrial premature beat
Prolonged QT interval

Submit Back to list

You can move the green and blue lines to measure the time interval in any part of the record.

Summary View

MRN: MRN-bd5f2e2f-66771
Name: Colton Berge
Recorded: 2018-05-18 13:55:30 UTC-04:00
Uploaded: 2021-02-02 05:14:00 UTC-04:00

Gender: Male
Age: 25yrs
Date Of Birth: 07/05/1992 (mm/dd/yyyy)

Ven. Rate: 120 bpm
PR Interval: 171 ms
QRS duration: 105 ms

QT/QTc interval: 371/428 ms
QTcF: -- ms
P, QRS, T axis: 70, 70, 55

Caliper

Chang : 2021-02-20 10:05:46 UTC+00:00
Normal sinus rhythm
Couplet atrial premature beat
Prolonged QT interval

Preliminary Interpretation : 2021-02-20 23:05:15 UTC+13:00
Normal sinus rhythm
Couplet atrial premature beat
Prolonged QT interval
Not Confirmed by Physician

Doctor's Comment

Doctor's Name

Normal sinus rhythm
Couplet atrial premature beat
Prolonged QT interval

Lead	I	II	III	aVR	aVL	aVF	V1	V2	V3	V4	V5	V6	Rhythm	ms
Diff.	1000	1000	1000	1000	1000	1000	1000	1000	1000	1000	1000	1000	3000	ms

The time interval will be measured.

QTM Logout

Summary View

MRN MRN-bd5f2e2f-66771 Gender Male Ven. Rate 79 bpm
Name Colton Berge Age 25yrs PR Interval 171 ms
Recorded 2018-05-18 13:55:30 UTC-04:00 Date Of Birth 07/05/1992 (mm/dd/yyyy) QRS duration 105 ms
Uploaded 2021-02-02 05:14:00 UTC-04:00

QT/QTc interval 371/428 ms
QTcf -- ms
P, QRS, T axis 70, 70, 55

Caliper

Lead	I	II	III	aVR	aVL	aVF	V1	V2	V3	V4	V5	V6	Rhythm	ms
Diff.	1000	1000	1000	1000	1000	1000	1200	1000	1000	1000	1000	1000	3000	ms

25mm/sec 10mm/mV

Change : 2021-02-20 10:05:46 UTC-00:00
Normal sinus rhythm
Couplet atrial premature beat
Prolonged QT interval

Preliminary Interpretation : 2021-02-20 23:05:15 UTC+13:00
Normal sinus rhythm
Couplet atrial premature beat
Prolonged QT interval
Not Confirmed by Physician

Doctor's Comment

Doctor's Name

Normal sinus rhythm
Couplet atrial premature beat
Prolonged QT interval

Submit Back to list



5.3 ECG Diagnosis (physician account ONLY)

5.3.1 Preliminary Interpretation is shown on the summary view.

The screenshot displays the 'Summary View' of an ECG report. At the top, patient information is shown: MRN (MRN-bd5f2e2f-66771), Name (Colton Berge), Gender (Male), Age (25yrs), Date of Birth (07/05/1992), Ven. Rate (79 bpm), PR Interval (171 ms), QRS duration (105 ms), QT/QTc interval (371/428 ms), QTcF (ms), and P, QRS, T axis (70, 70, 55). The ECG traces are displayed in a grid format, with leads I, aVR, V1, V4, II, aVL, V2, V5, III, aVF, V3, V6, and II. A 'Caliper' tool is visible above the traces. On the right side, the 'Preliminary Interpretation' section is highlighted with a red box, containing the following text: 'Normal sinus rhythm', 'Couplet atrial premature beat', 'Prolonged QT interval', and '**Not Confirmed by Physician**'. Below this, the 'Doctor's Comment' section is also highlighted with a red box, containing a text area with the same text as the preliminary interpretation, a 'Submit' button, and a 'Back to list' button.

5.3.2 Final Interpretation.

Physicians can make annotations in the **"Doctor's Comment"** section.

The screenshot displays the 'Summary View' of an ECG report, similar to the previous one. The patient information and ECG traces are the same. The 'Preliminary Interpretation' section is visible but not highlighted. The 'Doctor's Comment' section is highlighted with a red box, showing a text area with the following text: 'Normal sinus rhythm', 'Couplet atrial premature beat', and 'Prolonged QT interval'. Below the text area, there is a 'Submit' button and a 'Back to list' button.

Final interpretation is shown on the summary view.

QTM Logout

Summary View

MRN MRN-1881b88c-23600	Gender Other	Ven. Rate 120 bpm	QT/QTc interval 371/428 ms
Name Hiram Orn	Age 24yrs	PR Interval 171 ms	QTcF -- ms
Recorded 1971-08-10 19:16:49 UTC+04:00	Date Of Birth 09/28/1995 (mm/dd/yyyy)	QRS duration 105 ms	P, QRS, T axis 70, 70, 55
Uploaded 2021-02-20 14:05:15 UTC+04:00			

Change 2021-02-20 10:05:46 UTC+00:00

- Normal sinus rhythm
- Couplet atrial premature beat
- Prolonged QT interval

Preliminary Interpretation : 2021-02-20 13:05:15 UTC+13:00

- Normal sinus rhythm
- Couplet atrial premature beat
- Prolonged QT interval
- **Not Confirmed by Physician**

Doctor's Comment

Doctor's Name


Normal sinus rhythm
Couplet atrial premature beat
Prolonged QT interval



6. Patient Profile Management


6.1 Overview

6.1.1 Click **"Patient profile"**, you can see all uploaded patient profile from the dashboard.

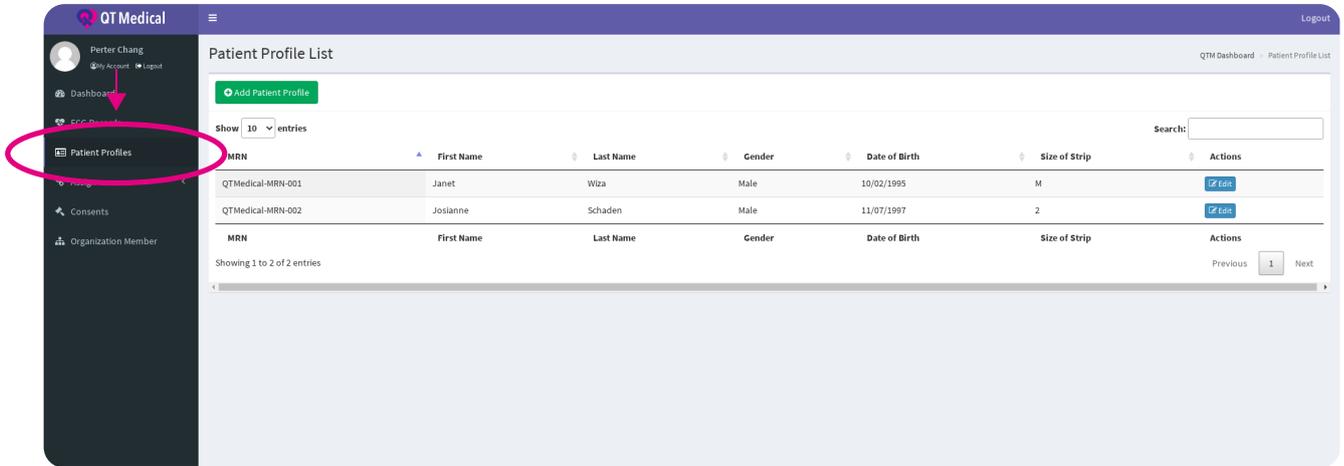

6.1.2. Add Patient Profile

Click **"Add Patient Profile"** button.

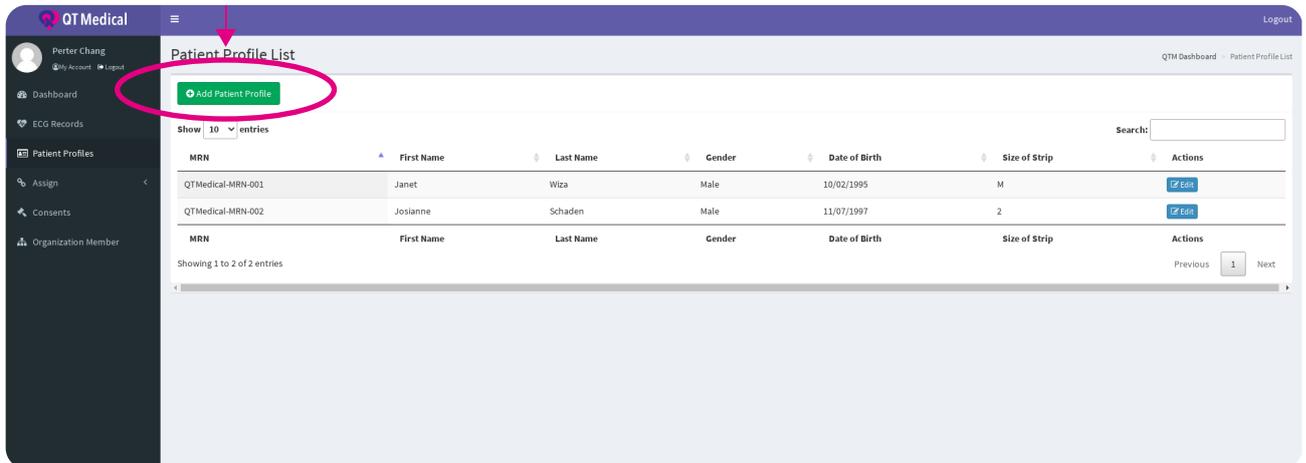

Fill in the patient profile and Click "Create".

QT Medical

Perter Chang

Dashboard

ECG Records

Patient Profiles

Assign

Consents

Organization Member

Create Patient Profile

QTM Dashboard - Patient Profile List - Create Patient Profile

* MRN
MRN

* First Name
First Name

* Last Name
Last Name

* Gender
Please choose your gender

* Date of Birthday
mm/dd/yyyy

* Size of Strip
Please choose your size

Create

The patient profile is created.

QT Medical

Perter Chang

Dashboard

ECG Records

Patient Profiles

Assign

Consents

Organization Member

Patient Profile List

QTM Dashboard - Patient Profile List

Add Patient Profile

Show 10 entries

Search:

MRN	First Name	Last Name	Gender	Date of Birth	Size of Strip	Actions
QTMedical-MRN-001	Janet	Wiza	Male	10/02/1995	M	Edit
QTMedical-MRN-002	Josianne	Schaden	Male	11/07/1997	2	Edit
QTMedical-MRN-003	Jimmy	Lu	Male	09/29/1990	XL	Edit


Showing 1 to 3 of 3 entries

Previous 1 Next




6.1.3 Edit Patient Profile

Click the **"Edit"** button on the right.

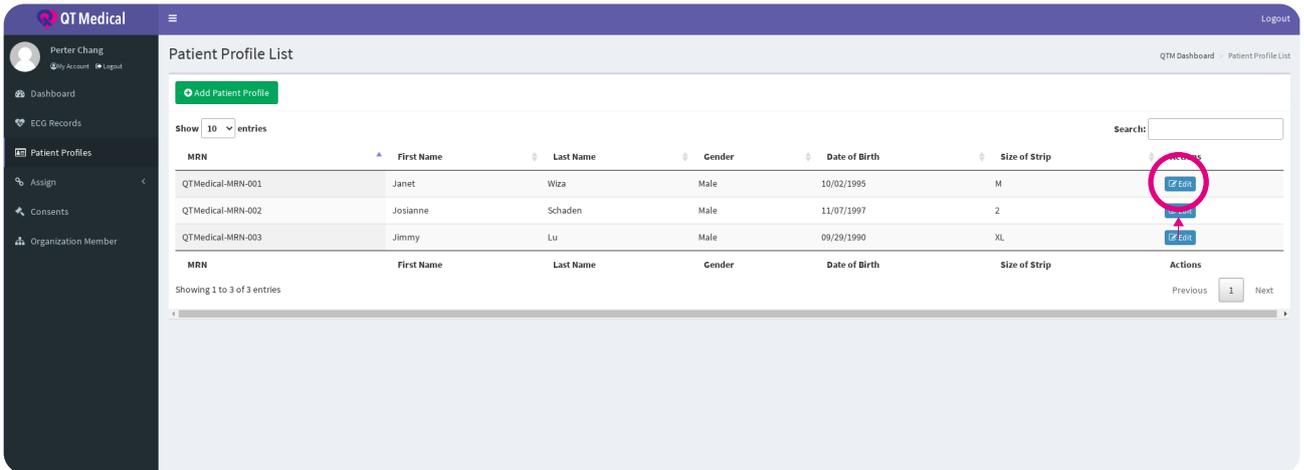

Start editing and click **"Update"** to save.

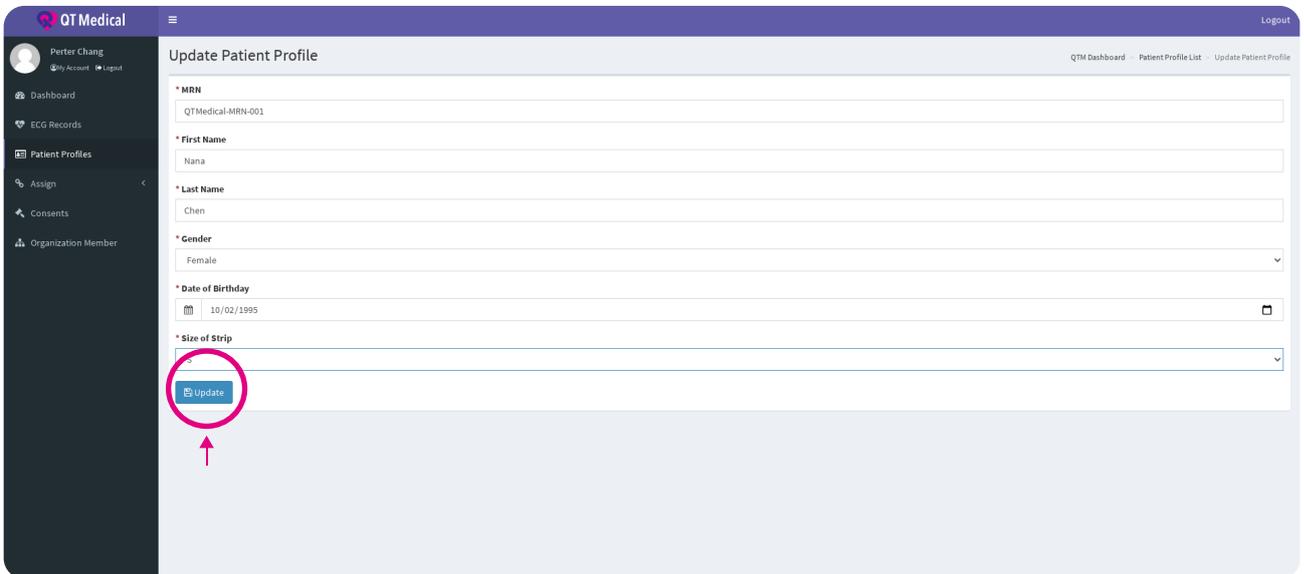

7. Assign

7.1 Patient Profile to ECG

Click **"Assign"**, select **"Assign ECG to Patient"**. You will see the list by session.

The screenshot shows the 'Assign ECG to Patient' interface. The left sidebar contains navigation options: Dashboard, ECG Records, Patient Profiles, Assign, Assign ECG to Patient, Consents, and Organization Member. The 'Assign' and 'Assign ECG to Patient' options are highlighted with red circles and arrows. The main content area displays a table of ECG sessions. The table has the following columns: Session, Patient, Time Zone, Session Recording Started, Operator, Notes, and Actions. The data rows are as follows:

Session	Patient	Time Zone	Session Recording Started	Operator	Notes	Actions
564e51a0	Not Specified	Antarctica/Davis	04/01/1995 21:25	Anthony Xu	Detail	Reassign
1c95a720	Lavern Watsica	America/Argentina/Buenos_Aires	02/07/1993 21:40	Anthony Xu	Detail	Reassign
f109a009	Lavern Watsica	America/Kentucky/Monticello	10/18/2004 09:11	Anthony Xu	Detail	Reassign
f109a009	Lavern Watsica	Indian/Cocos	10/18/2004 19:41	Anthony Xu	Detail	Reassign

Showing 1 to 4 of 4 entries

ECG Records created by Guest mode will be displayed as "Not Specified" and will need to be reassigned to the correct patient.

The screenshot shows the 'Assign ECG to Patient' interface. The left sidebar contains navigation options: Dashboard, ECG Records, Patient Profiles, Assign, Assign ECG to Patient, Consents, and Organization Member. The 'Assign ECG to Patient' option is highlighted with a red circle. The main content area displays a table of ECG sessions. The table has the following columns: Session, Patient, Time Zone, Session Recording Started, Operator, Notes, and Actions. The data rows are as follows:



Session	Patient	Time Zone	Session Recording Started	Operator	Notes	Actions
564e51a0	Not Specified	Antarctica/Davis	04/01/1995 21:25	Anthony Xu	Detail	Reassign
1c95a720	Lavern Watsica	America/Argentina/Buenos_Aires	02/07/1993 21:40	Anthony Xu	Detail	Reassign
f109a009	Lavern Watsica	America/Kentucky/Monticello	10/18/2004 09:11	Anthony Xu	Detail	Reassign
f109a009	Lavern Watsica	Indian/Cocos	10/18/2004 19:41	Anthony Xu	Detail	Reassign

Showing 1 to 4 of 4 entries





Click "Reassign", select the patient profile list.

When assigned successfully, you will see the success message popup.

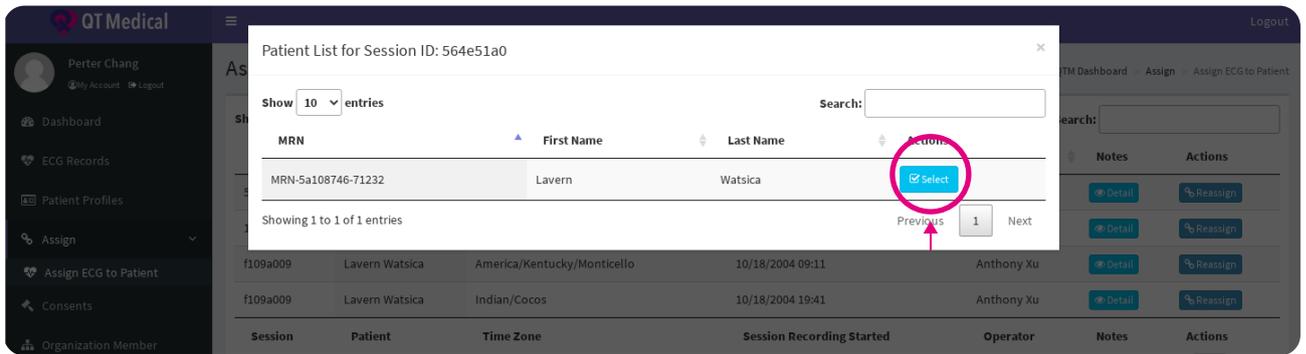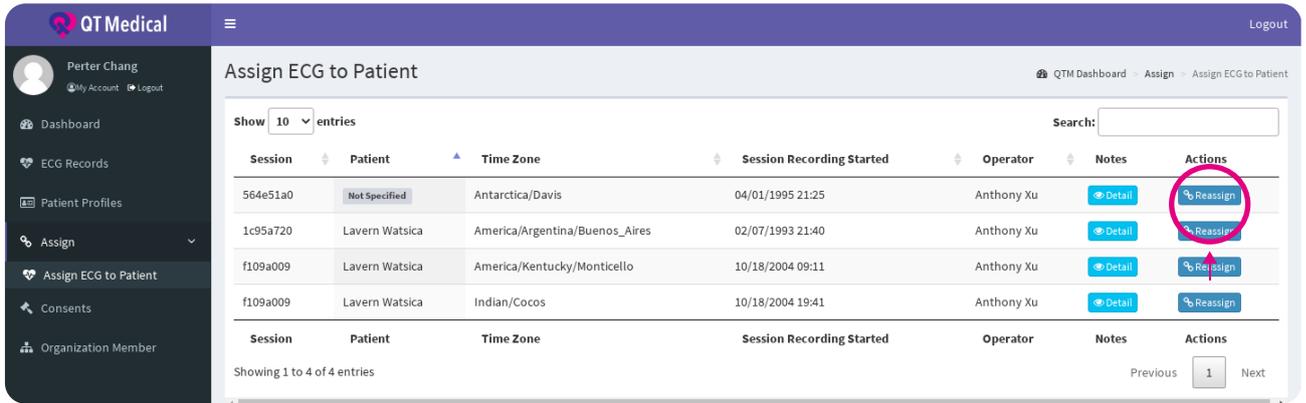

And you can click the "Detail", you can see details of Note.

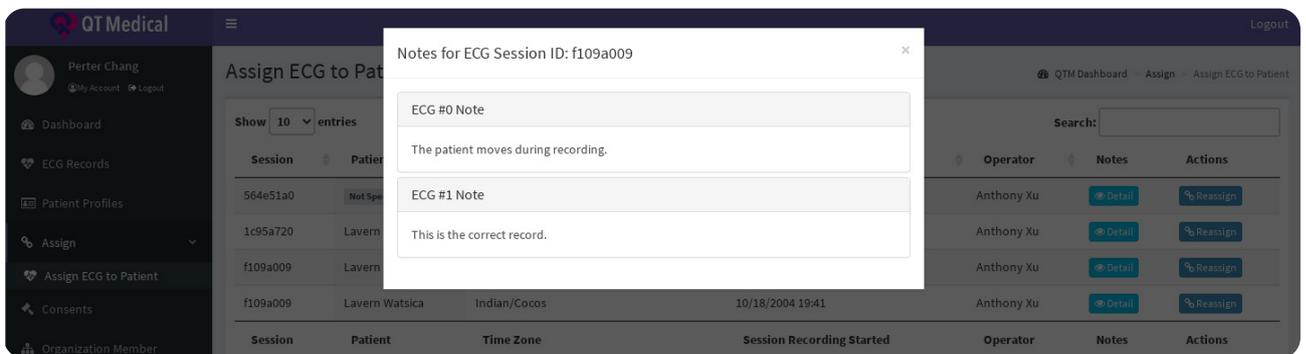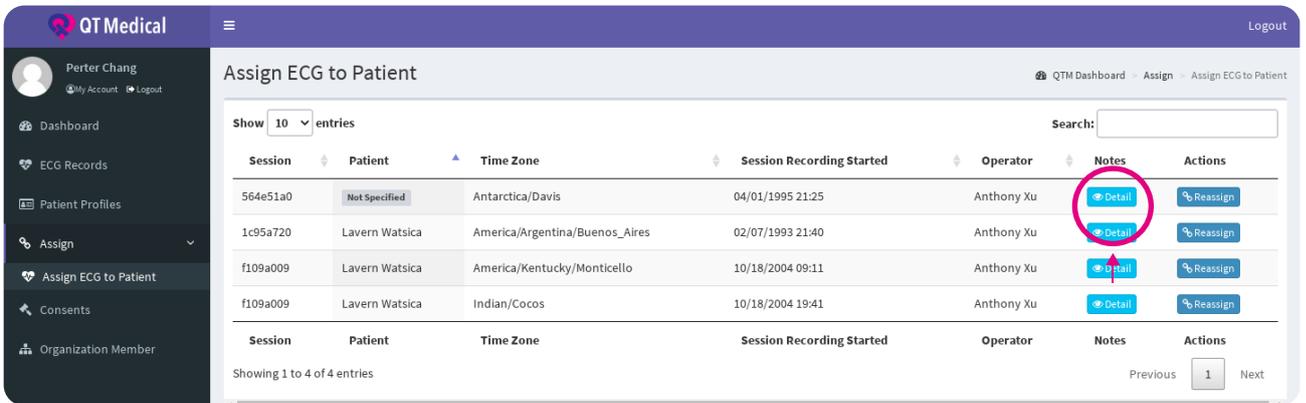

Customer Service Support

For customer service support, please contact:



COBS Pte Ltd

196, Pandan Loop,
#06-29 (Suite 18)
Pantech Business Hub,
Singapore 128384

T: +65 8022 8433

E: hello@ecg-future.com